

# Quick Start: QPC XSCAN32 Chiro Templates

## Chiro Templates


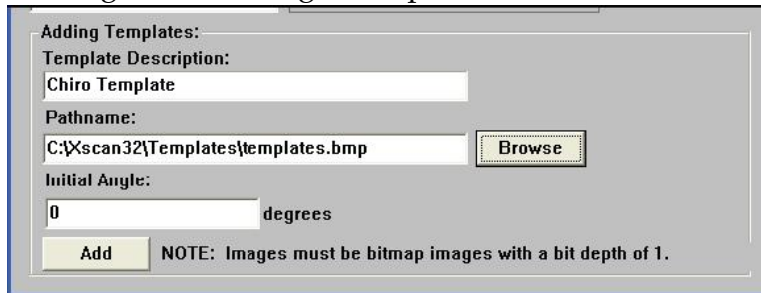
1. Open an image for viewing and click on the **Template** button in the *Annotations* tool bar.  
 **Note:** If the *Annotations tool bar* is not open, go to the *View* menu and select **Annotations and Measurements**.
2. To add a new template enter a Template Description, Pathname, Initial Angle and then click the **Add** button. The template will be added to the list of available templates.

Figure 1: Adding a Template in QPC XSCAN



Adding Templates:

Template Description:  
Chiro Template


Pathname:  
C:\Xscan32\Templates\templates.bmp

Initial Angle:  
0 degrees

NOTE: Images must be bitmap images with a bit depth of 1.

3. To view an existing template, select the template from the Template list. It will appear in the view box.
4. Click the **Load** button and the template will be placed as an overlay on the currently opened image. Move the mouse around to position the template on the image. Left mouse click on the image to accept the position.


Figure 2: Loading a Template in QPC XSCAN

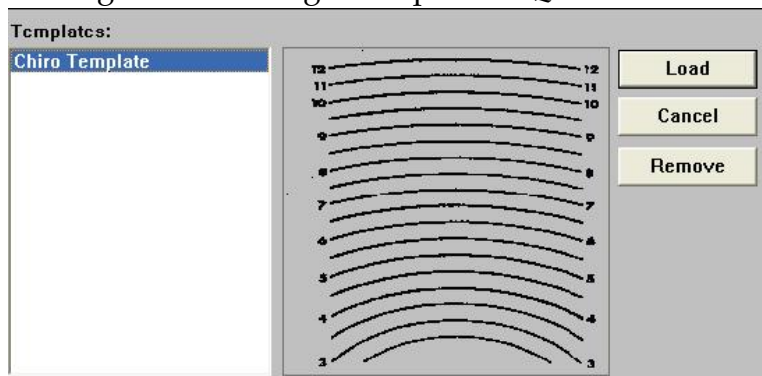

# Quick Start: QPC XSCAN32 Chiro Templates

5. To move the image to a new position, left mouse click on the center icon and drag the mouse around to position the template on the image. When you release the mouse it will be positioned on the image.

Figure 3: Moving a Template in QPC XSCAN

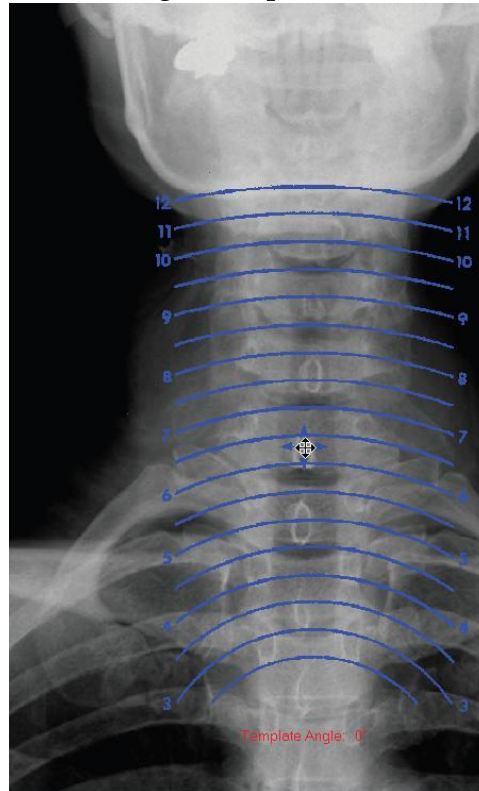

6. To rotate the image left mouse click on the center icon, In each corner of the template there are rotating icons. Left mouse click on any of the rotating icons and drag the template around. The angle will change accordingly. When you release the mouse it will be positioned on the image.

Figure 4: Rotating a Template in QPC XSCAN

