

# Quick Start: QPC XSCAN32 Customizing Field Labels

## Customizing Field Labels


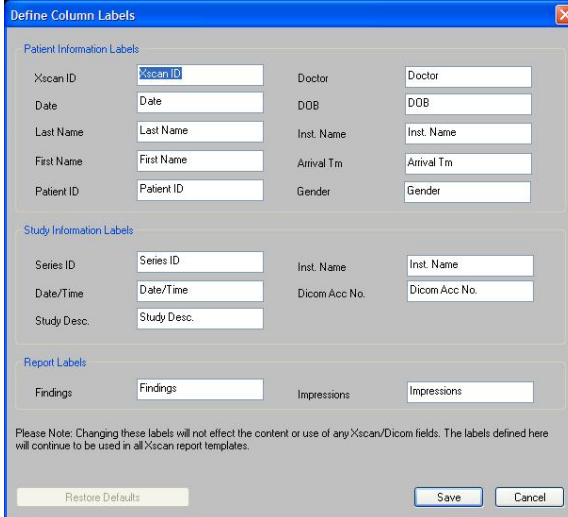
1. Open QPC XSCAN32, then go to *Options* → *Define Column Labels*.
2. Change the default column labels as appropriate.  
 **Note:** Some fields are designated for text and others dates.
3. Click **Ok** to confirm the changes. The *Patient/Exam Information* window will be updated and the changes applied.

Figure 1: Defining Custom Labels in QPC XSCAN



Define Column Labels

Patient Information Labels

Xscan ID	Xscan ID	Doctor	Doctor
Date	Date	DOB	DOB
Last Name	Last Name	Inst. Name	Inst. Name
First Name	First Name	Arrival Tm	Arrival Tm
Patient ID	Patient ID	Gender	Gender

Study Information Labels

Series ID	Series ID	Inst. Name	Inst. Name
Date/Time	Date/Time	Dicom Acc No.	Dicom Acc No.
Study Desc.	Study Desc.		

Report Labels

Findings	Findings	Impressions	Impressions
----------	----------	-------------	-------------

Please Note: Changing these labels will not effect the content or use of any Xscan/Dicom fields. The labels defined here will continue to be used in all Xscan report templates.

Restore Defaults Save Cancel

4. The User may restore the default labels by selecting the **Restore Defaults** button.

### Default Patient column labels:

Xscan ID	Date	Last Name	First Name	Patient ID	Doctor	DOB	Inst. Name	Arrival Tm
----------	------	-----------	------------	------------	--------	-----	------------	------------

### Default Exam column labels:

Exam ID	Exam Date	Study Desc.	Inst. Name	Dicom Acc No
---------	-----------	-------------	------------	--------------