

Quick Start: QPC XSCAN32 Keyboard Shortcuts

QPC XSCAN32 Keyboard Shortcuts

Function Keys

F1	Rotates an image clockwise.
F2	Rotates an image counter-clockwise.
F3	Horizontally flips an image.
F4	Vertically flips an image.
F5	Inverts an image.
F7	Opens and closes the Window/Level dialog.
F8	Opens and closes the CR Processing dialog.
F9	Resets the window/level of an image to the original window/level setting.
Shift + F8	Opens and closes the Annotations dialog.
Ctrl + F8	Turns the patient information overlay on and off.
F11	Opens QPC XScan32 Help.
F12	Selects and deselects all image icons.
Ctrl + A	Selects and deselects all image icons.
Ctrl + D	Deletes the selected image icons.
Shift + P	Opens preferences.
Shift + S	Adds images to the send queue.
Ctrl + S	Sends queued images.
Shift + Q	Adds images to the send queue and sends the queued images.
Shift + R	Imports received images immediately.
Right Arrow	Moves to the next image.
Left Arrow	Moves to the previous image.
Down Arrow	Moves to the next row of images.
Up Arrow	Moves to the previous row of images.
Page Down	Moves to the next page.
Page Up	Moves to the previous page.

