
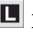




Quick Start: QPC XSCAN32 CR Markers

Left and Right CR Markers

1. Open an image for viewing.
2. Select the Left/Right Markers  button to apply a Left or Right marker to the image.
3. Select either the Left or Right marker. When the user clicks on the Left/Right Marker button the Left  marker icon will appear. If the user clicks the Left/Right Marker button again, the Right  marker icon will appear. Clicking again will toggle back to the Left marker.
4. Move the mouse onto the part of the image where the marker needs to appear. When the user moves the mouse over the image a Left or Right marker icon will appear in place of the mouse.
5. Click on the image to place the icon.
6. To remove the marker click on the Left/Right Marker  button again.

