
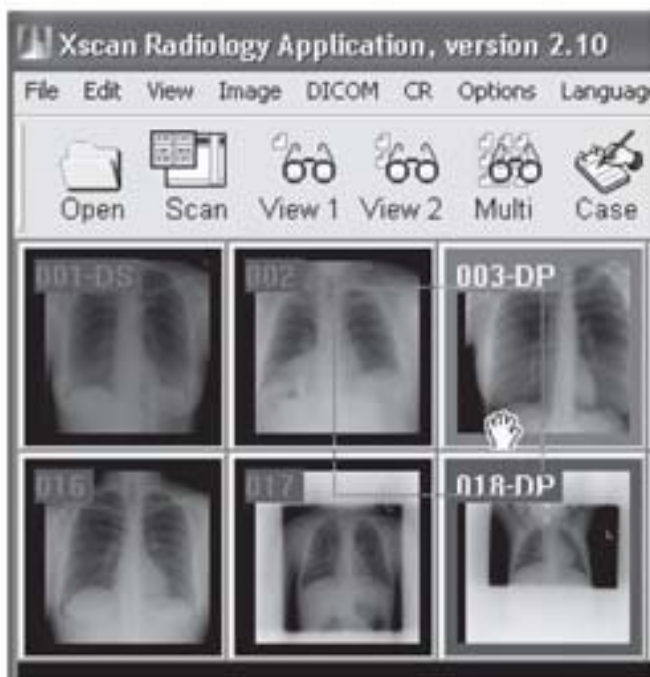



Quick Start: QPC XSCAN32 Moving Icons

Moving Icon Thumbnails

1. Open a patient study for viewing.
2. Hold down the Ctrl key while clicking the left mouse button on the image that needs to be moved. The arrow icon will change to a hand icon . The icon highlighted in red shows the location where the image will be dropped.
3. To drop the icon, simply release the left mouse button.



 **Note:** If the moving icon is highlighted, this indicates that it will return to its original position. If an icon is dragged forwards in position (i.e. the image number of the highlighted icon is higher than the moving icon) then all icons between its original position and its new position will move backwards and each image number decreases by one. If an icon is dragged backwards in position (i.e. the image number of the highlighted icon is less than the moving icon) then all icons between its original position and its new position will move forward and each image number increases by one. If no icons are highlighted when dragging, then the image will be returned to its original position.