
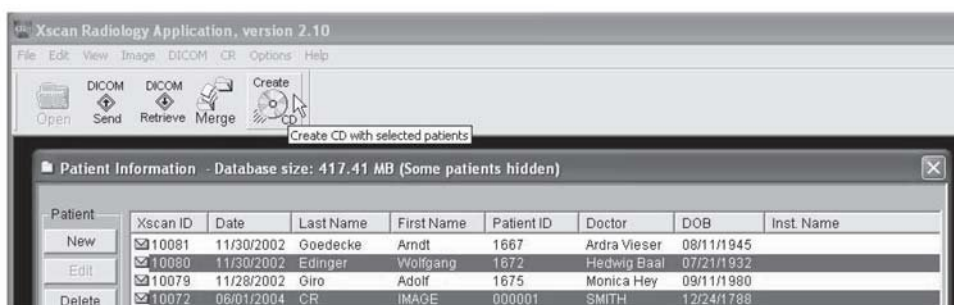



Quick Start: QPC XSCAN32 CD/DVD Creator

Patient CD/DVD Creator

 **Note:** CD/DVD Creation Software is *NOT* part of the QPC XSCAN32 Software. This section is optional for users who already have CD/DVD Creation Software already installed on their respective PCs.

1. In the patient information window select the patient(s) to be placed on the CD/DVD. To select multiple patients hold down your Ctrl key on the keyboard and using your left mouse button select each patient.
2. Click on the Create CD button in the toolbar. Selected patients will be copied into the patient CD layout folder (C:_XscanCDFolder).



 **Note:** If this is the first time using the patient CD creator feature a prompt will appear indicating the user to browse for the location of the DVD/CD Creator software. Alternatively, if Nero software is already installed on the PC, select the option *Use Nero scripting to create CD/DVD*.




Using Nero Scripting

1. When the software has finished initializing and copying the patient CD, the Nero scripting dialog will appear.
2. Select the device to burn to. (if multiple drives are present)
3. Put the CD/DVD into the drive and select the **Burn** button.


Quick Start: QPC XSCAN32 CD/DVD Creator

4. The drive will eject media once it has completed. Click the **Burn** button again to make multiple copies (or alternatively set the Number of copies option before burning).

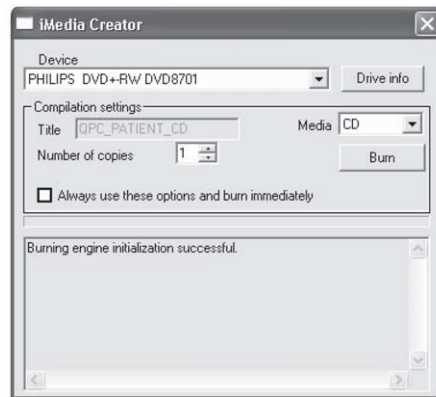


Using other CD/DVD recording software

1. When the software has finished initializing and copying the patient CD, a 'Ready to create CD' prompt will appear, select **Yes**.
2. The DVD/CD Creator software will open. Select source folder `C:_XscanCDFolder` and drag the contents of the Xscan CD Folder folder and two files into the data project region.

 **Note:** The user must not create their own folders on the CD/DVD. The autorun.inf file must appear in the top level (under the root directory) of the CD/DVD.

Select source folder
location `C:_Xscan
CDFolder`

The following folders
and files are to be
copied onto a
DVD/CD:
* Xscan folder
* xscan.bat
*Autorun.inf

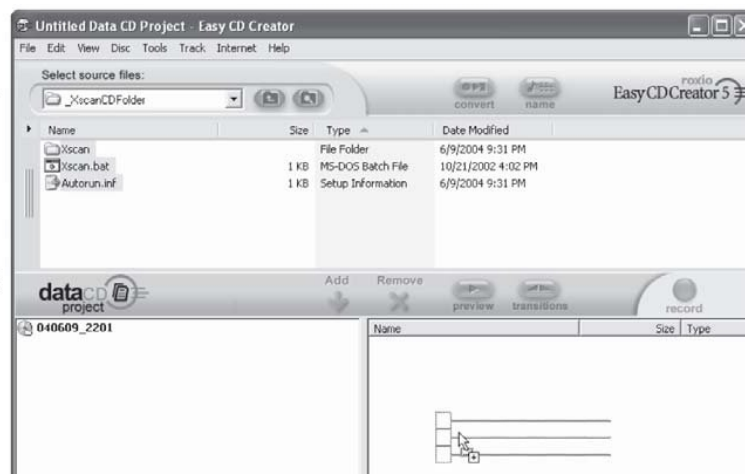

3. Click the Record button in CD Creator software.